

Push Life's Left Sided Wheelchair Lever

A20 Wheelchair Assist

Ava Wagstaff - 300008925, Opeoluwa Onipe - 300043785, Thomas Tokarczyk - 300021948


Push Life's left sided wheelchair lever is here to help you. It is built to give you more mobility on a daily basis and allows you to truly meet your full potential.

Materials Supplied

- Gear double axle system
- Bearing (x2)
- Spacer
- Lever Mechanism
- O-Rings (x5)
- Aluminum Support (x2)
- Ring Clamps (x4)
- Spring

Installation Instructions

1. Please place the wheelchair on it right wheel on a flat surface
2. Then place the lever on the center axle of the left wheel, with larger hollow axle making contact with the wheel axle and spokes, while the thin axle is on the outside of the wheelchair. For it to look as follows in the future:

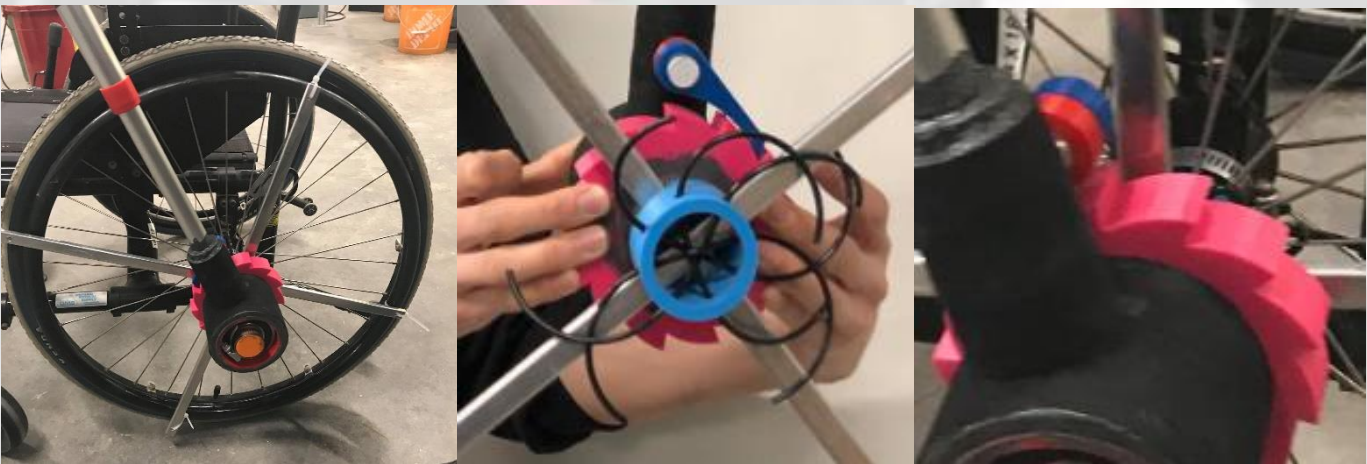

3. The feed four o-rings through the hollow axle, please pull the o-ring through the holes across from one another
4. Then feed the o-ring ends through the spokes and clamp then down tightly, with the ring clamp provided, on the inner axle of the wheel
5. Then insert the aluminum supports into the hollow axle, into the holes across from one another, ensure even length are out on both sides, then zip tie the supports to the rim of the wheelchair



6. Then please place the bearings and spacers on the thin axle in the order as presented above, bearing spacer bearing
7. Please then insert the lever mechanism onto the axle with bearings and spacer, clamping the axle off with another ring clamp



8. Please Place 2 ring clamps with the spring attached to a desired position on the wheelchair lever and the wheelchair frame itself, to allow for the self return of the lever
9. The lever is now in working order and will allow to further your mobility capacities

Use

- Exercise caution while using the lever
- Start with a lighter pressure on the lever, and adjust the spring until an ideal tension is found
- Adjust height of lever for comfort purposes
- And push your way into a more mobile future

Warnings

Please understand that the following product is meant to enhance mobility. It is meant to be used within reason, safe and level-headed movement should always be considered by the user. The lever is only meant to allow for more comfortable mobility, no safety implications have been made, and all safety precautions of the wheelchair user is on should still be taken into consideration.

Please keep the lever well maintained, handle with care, and upkeep and clean the lever regularly to ensure the longevity of the lever.

**Let us give you that PUSH forward
into a more MOBILE future!**